
## **ORTHOTICS-ONLINE**

## LIMITED

## How to fit Metatarsal Barss in your shoes

- 1. Find the widest part of shoe, see dotted line on diagram.
- 2. Fit Met Bar just behind widest part and as indicated.
- 3. Carefully trim sides with sharp scissors if necessary.
- 4. Peel off backing and fix in place.
- 5. This product needs trainers or roomy shoes.

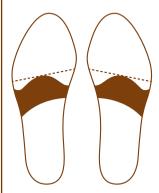

A simple way to find the correct position for you is to try out your Met Bar by attaching with blu-tack, moving if necessary until you find the best position, then peel and stick in place. N.B. Your Met Bards are marked LEFT and RIGHT.